



# SmartSoc BYOD – New Device Setup

## Setup a new device for BYOD: How to


1. Acquire an approved BYOD mobile device. - Ref. *SmartSoc BYOD - Approved Device Models (SL00266 Rev A)*
2. Establish an Apple ID for the device (**REQUIRED**) using a valid email address.
  - a. Using a computer with internet connection, go to the website <https://appleid.apple.com/account>

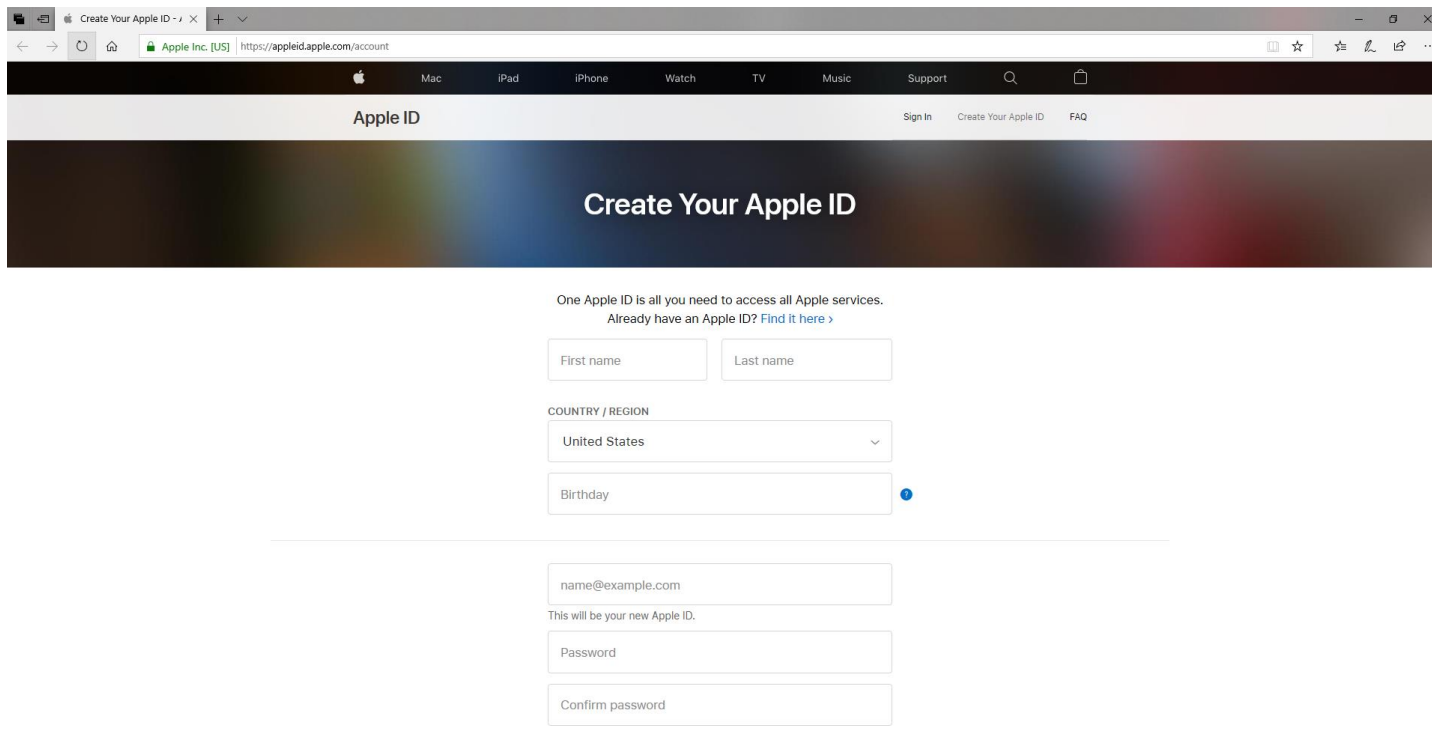


- b. Fill out the personal information for the Apple ID account.

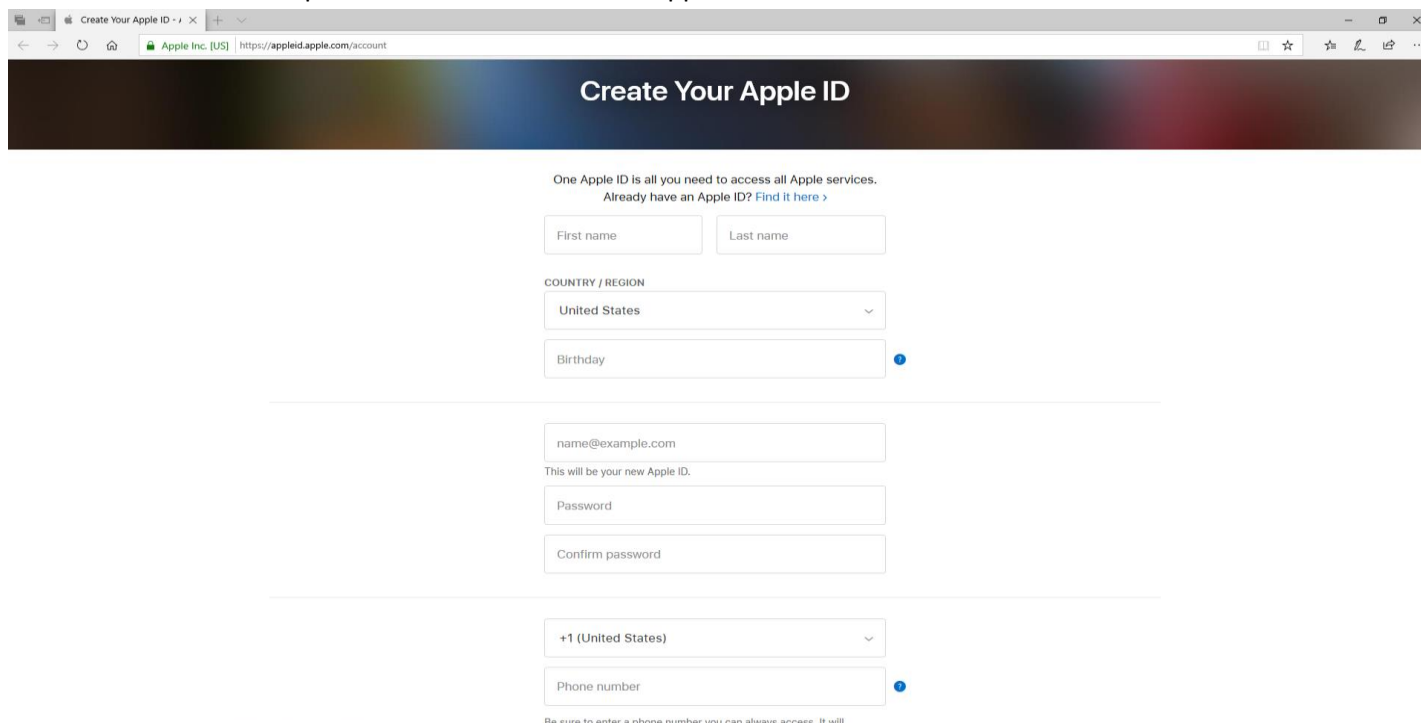

c. Select "Continue" upon successfully filling out all required fields.

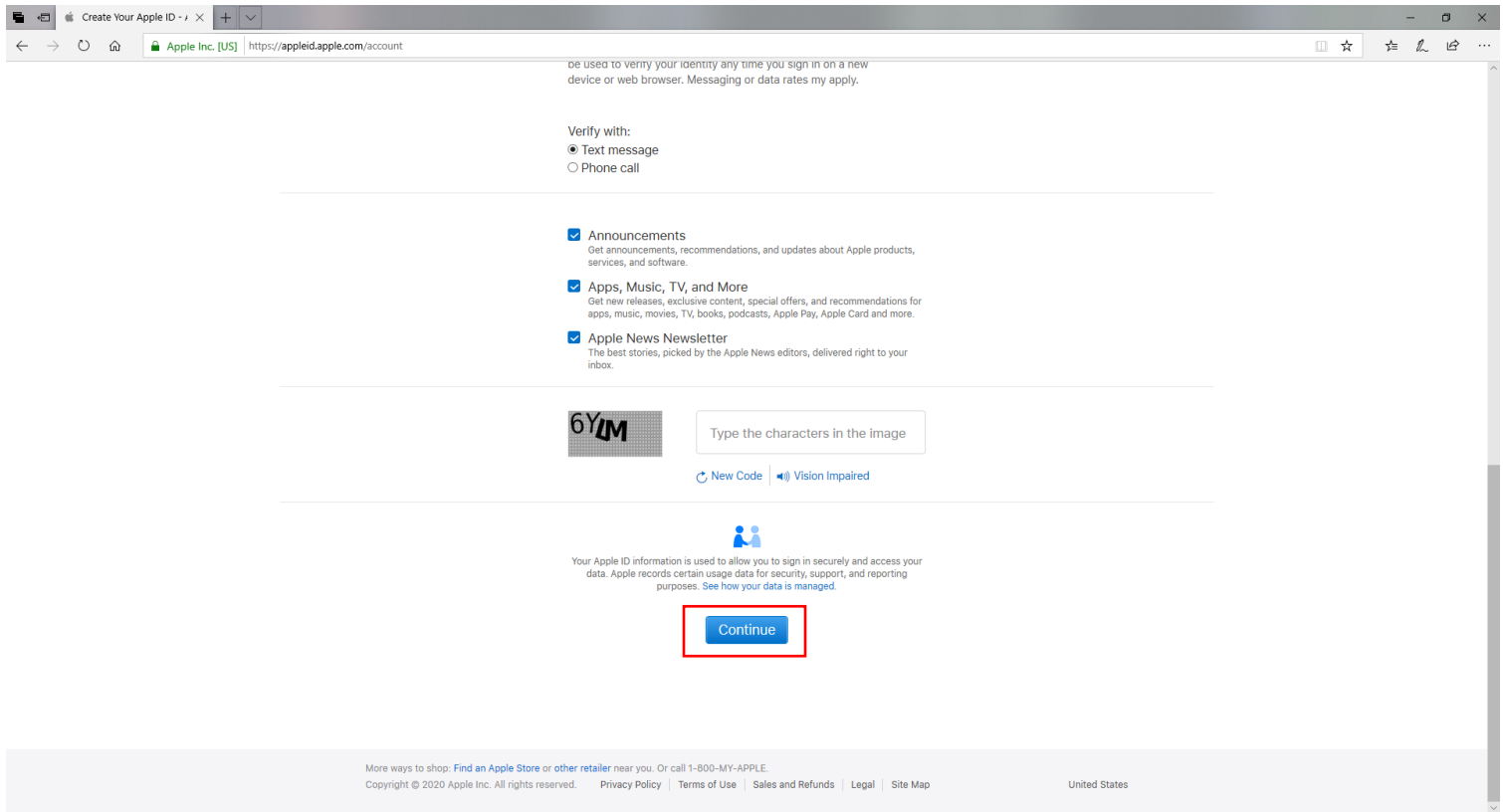


d. Verify the Apple ID email address and phone number.

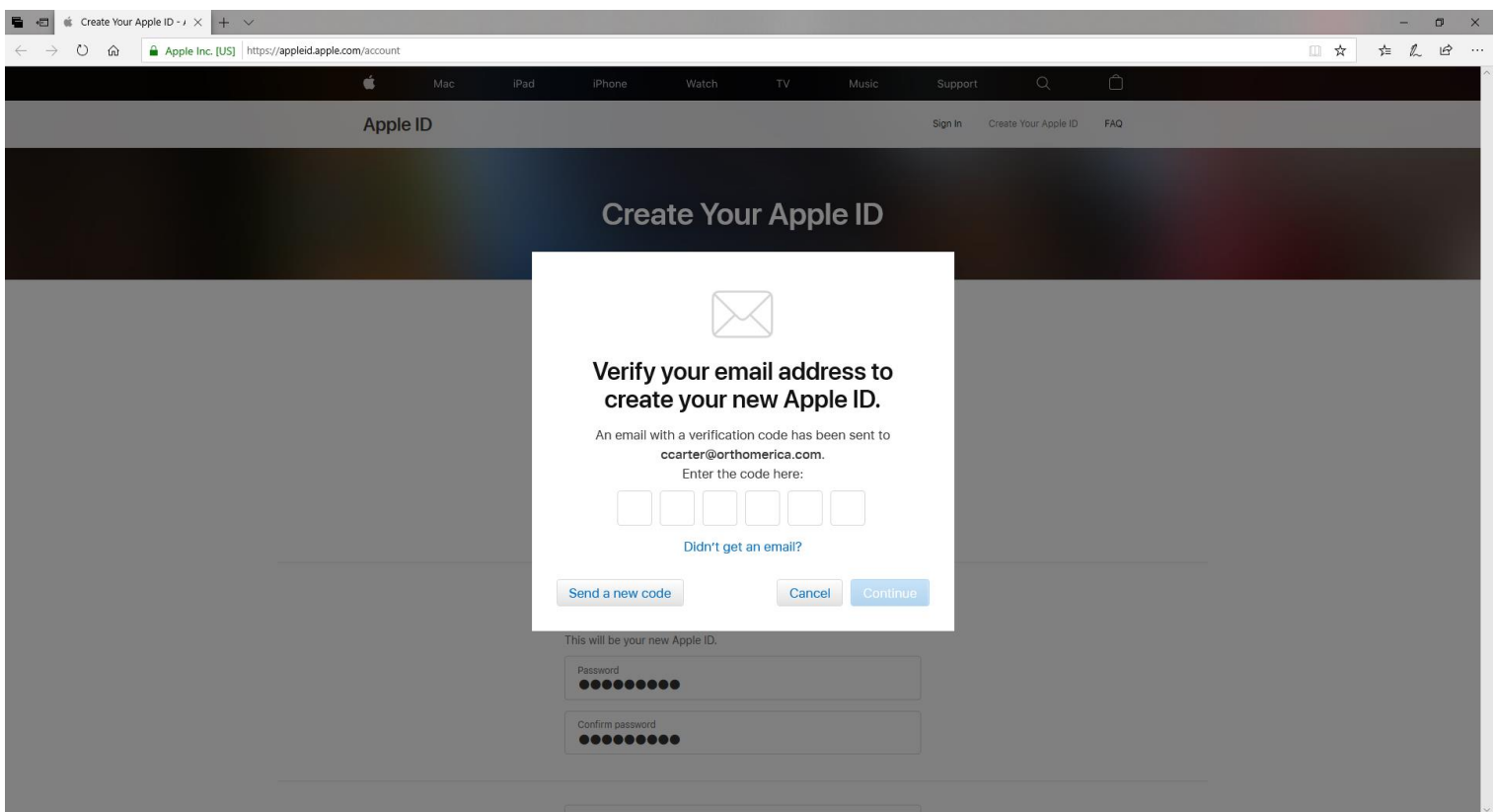

e. Upon verifying the account, the Apple ID will be established.

3. Setup your iOS iPhone 11(s) device.
  - a. Power on your new device by holding down the large button on the right edge of the device.

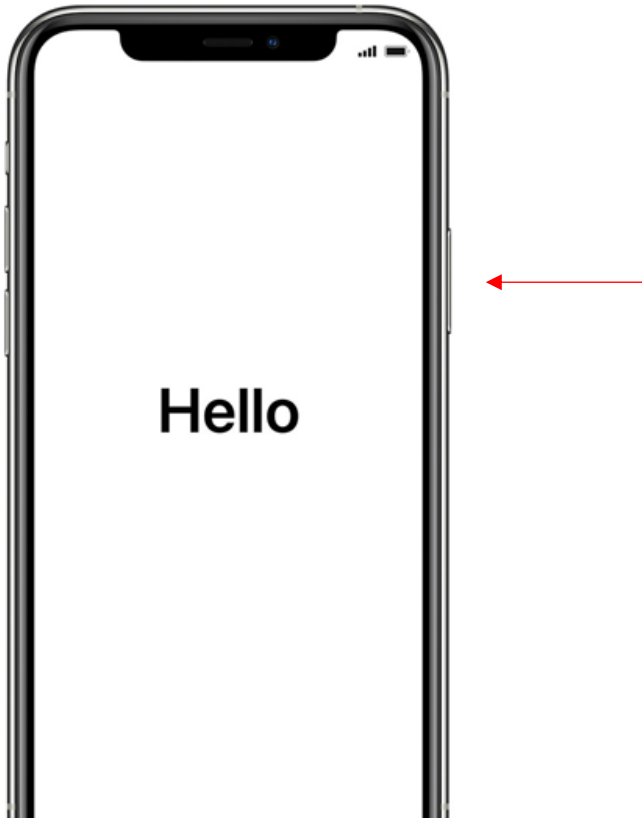

- b. Select language.



c. Select country.




d. Select "Set Up Manually".




- e. Choose a Wi-Fi network connection.




- f. Set up Face ID and/or Touch ID, or select "Set Up Later in Settings".

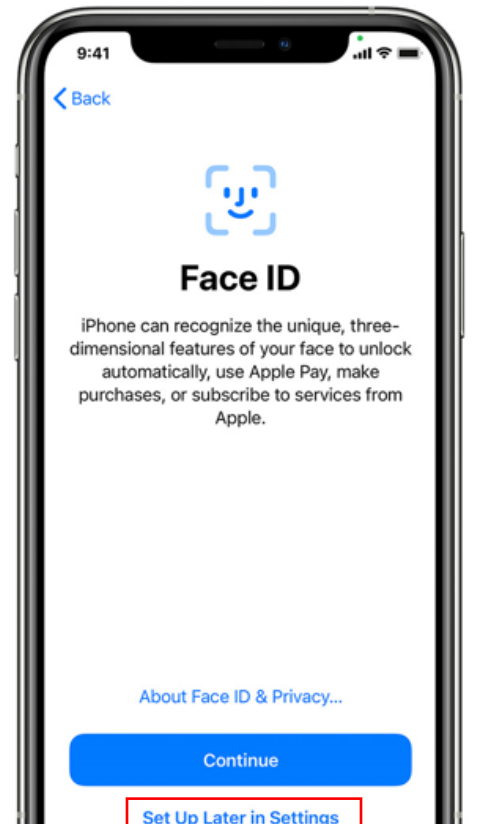

- g. Create a passcode for the device.

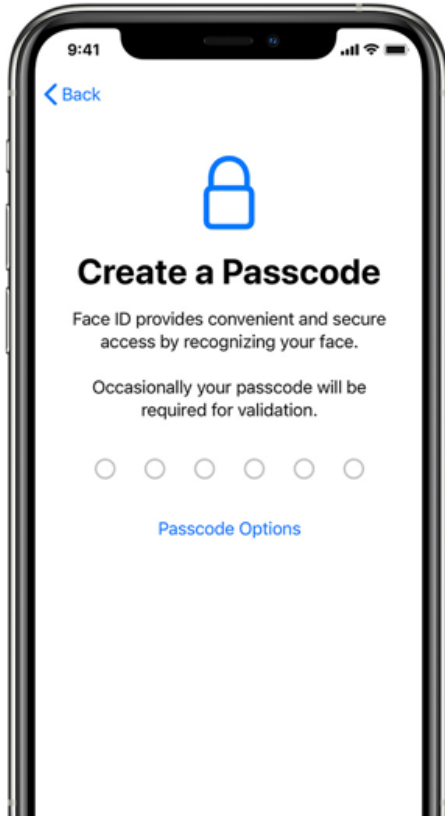

- h. Restore or transfer your information. If this is your first iPhone, select “Don’t Transfer Apps & Data”.

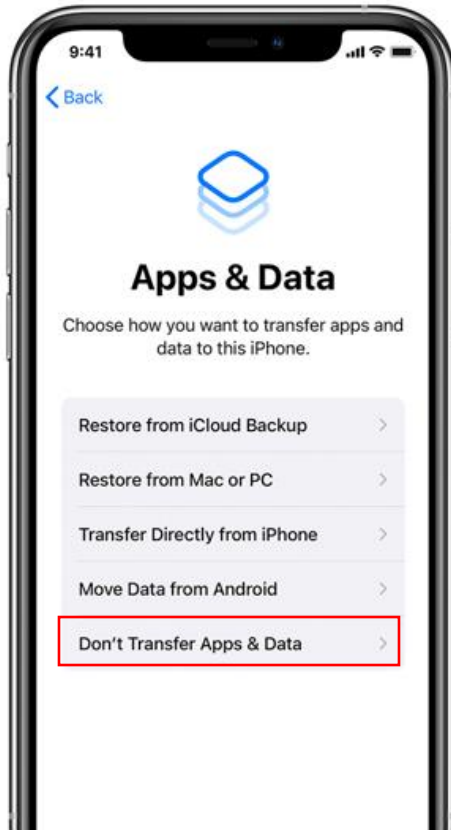

- i. Sign in with your created Apple ID (ref. Step 2).




- j. Turn on automatic updates and set up other personal features.



- k. Set up Siri and other phone aided services, or select “Set Up Later in Settings”.



- l. Setup Screen time and other display options, or select “Set Up Later in Settings”.

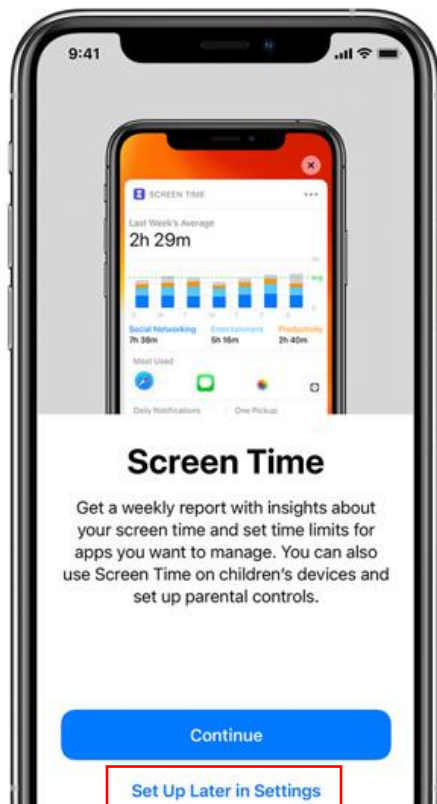

- m. Select “Finish Up” to finalize your device setup.